Salvafix®

FLAME RETARDANT

! WARNING

EN

READ & UNDERSTAND INSTRUCTIONS MANUAL BEFORE USING SALVAFIX PRODUCTS. FAILURE TO FOLLOW INSTRUCTIONS MANUAL COULD CAUSE SERIOUS INJURIES OR PATIENT'S DEATH.

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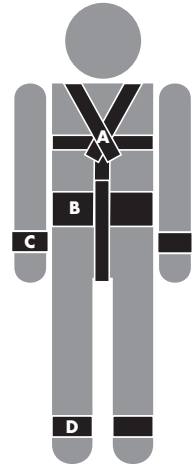
ENGLISH

INSTRUCTIONS FOR USE:

Please refer to the illustrations in this leaflet.

SAFETY RECOMMENDATIONS:

1. Use the size of product most suitable for the Patient (please see measurements table).
2. Please fully check the ENTIRE system BEFORE USE.
3. Firmly fix all Salvafix straps to the frame of the bed. In articulated beds always attach them to the moving frames.
4. ALWAYS use the two side bands of the Salvafix belt.
5. Use Salvafix waist belt with PERINEAL band in order to prevent patient from sliding up and down.
6. The TWO side guardrails of the bed must be always kept in the UPPER position and BLOCKED.
7. Ensure that the guardrails of the bed are just one part.
8. Constantly monitor all Patients that are being restrained by means of Salvafix mechanisms.
9. Follow all the instructions contained in the Restraint Protocol of your place of work and according the laws of your country.
10. DO NOT USE Salvafix if you have not been fully informed and trained in its use.
11. A quick and safe cutter should always be on hand for emergency cases.
12. The magnetic keys MUST NEVER be taken nearer than 1.5cm to Patients with a pace-maker.
13. Do not submerge and avoid liquids into the lock buttons **SVF3500 - SVF3510**.
14. Salvafix products must always be used with the locking system (buttons and pins) included in your package.

- DO NOT USE Salvafix products if you do not comply with ALL these points.
- Improper use of Salvafix restraint systems can cause serious injury or even death by strangulation or abdominal asphyxia.
- Manufacturer would not be liable when any of this recommendations are not met.

MAINTENANCE INSTRUCTIONS:

- "Salvafix FR" products are manufactured with flame retardant webbing.
- Before washing / drying Salvafix restraint products, the Salvafix locking buttons and anchoring pins should be removed.
- Salvafix products are made of 100% polyester and are resistant when washed at high temperatures (90°)
- Clean the Velcro hook side closure regularly of fluff and threads with a hard brush.
- Laundry bag is highly recommended to prevent loss of components during product washing and drying.

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